

## User guide

### Medi-Dised

Temporary filling, Radiopaque ,disinfection

#### Composition:

Barium Sulfate,Calcium hydroxide,Menthol, Iodoform,Camphor,Cresol, excipient .

#### Applications:

Antibacterial dressing in nonvital teeth

Prevention of internal root resorptio.

Prevention of inflammetory apical resorption followed by trauma.

endodontic treatment of reimplanted avulsed teeth

Due to its high pH above 12.5, Medi-Dis/Paste has a strong bactericidal effect

High Radiopacity

Creamy consistency also allows the paste to be efficiently placed in the root canal.

#### Storage

Store at 5-15°C

#### Warnings:

Corrosive. Consult MSDS for more information.

#### Package:

1- 15 ml

Art. No.10016-15

2- 30 ml

Art. No.10016-30



#### Swiss MediTec

Gfellen, POBox 513

6162 Finsterwald,Switzerland

Tel: + 41 56 624 02 37, +41 79 482 32 98

Fax:+ 41 56 624 02 37

[www.swiss-meditec.ch](http://www.swiss-meditec.ch)

[info@swiss-meditec.ch](mailto:info@swiss-meditec.ch)