

## User guide

### Medi-EBA

#### Root canal reinforced cement

#### Radiopaque

#### Composition:

**Powder** :Barium Sulfate ,Zinc Oxide,Thymol Iodide, Paraformaldehyde, Dexamethasone Acetate , Zinc Chloride,Bismuth trioxide,excipient .

**Liquid** :Eugenol,Ethoxybenzoic acid,excipient.

#### Indications

Treatment of perforations of root canal and furcation

Treatment of internal resorption

Retrograde root canal filling after apical resection

Pulp capping,Pulpotomy,Apexification,Apexogenesis

#### Dosage and administration:

##### Direction for linings

1-Place two scoops of powder on a clean dry glass slab or mixing pad.

2-Dispence one drop of liquid alongside the powder

3-Add up to 3/4 of the powder to the liquid and spatulate for 30 sec.using a stainless steel spatula.

4-Add the remaining powder and mix another 30 sec. Until thick putty like consistency is obtained.

5-Apply the mixture to the cavity with an amalgam plugger which has been dipped in water or alcohol.

6-Exposure can be directly capped with a thin mixture.Spray with warm water to accelerate setting time.

##### Direction for final cementation

1-Place four scoops of powder on a clean dry glass pad.

2-Dispence four drops of liquid alongside the powder.

3-Add up to 3/4 of the powder to the liquid and spatulate for 30 sec. Using a stainless spatula.

4-Add the remaining powder and mix another 30 sec. Until a smooth creamy mix is obtained.

5-Smear surrounding restoration to allow easy removal of excess cement.

6-Coat fitting surface and shoulder with a thin film of cement.Retain under steady finger presure until set.

#### Contra indications/Intractions:

Eugenol containing materials inhibits the polymerisation of acrylic direct filling materials (acrylic resins and composite resins) and temporary crown and bridge materials and use with these materials must be avoided.

#### Warnings:

Medi-ZOE Contains paraformaldehyde which can be toxic by inhalation or ingestion. Using gloves, mask and protective glasses strongly reccomended. In case of contact with the skin wash with plenty of soap and water.In case of skin irritation , get medical attention. In case of contact with the eye rinse cautiously with plenty of water for 10 minutes. In case of swallowing rinse the mouth with plenty of water, and consult physician. Do not force vomiting.

**Storage:** Store between 15°-25° C. Protect from direct light and humidity.


**Shel life:**Three years from date of manufacture .

#### Package:

**Full set** Art.No. 10016

**Liquid: 15 ml** Art.No.10017-1

**Powder: 20 gm** Art.No.10017-2

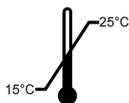

For dental professional use only .

Made in Switzerland.



**Swiss MediTec**

Gfellen, POBox 513,6162 Finsterwald,LU

Switzerland

Tel: + 41 56 624 02 37,

+41 79 482 32 98

[www.swiss-meditec.ch](http://www.swiss-meditec.ch)

[info@swiss-meditec.ch](mailto:info@swiss-meditec.ch)